

EntraFlo H2O™ Nutrition Delivery System

Quick Start Guide




Starting the pump:

1. Press **ON**. Previous pump settings stored.
2. If programming the pump for a new individual, press and hold **RESET** for 5 seconds when “Feeding Rate” is selected with the green light. Pump will beep and **1** will be displayed. The pump settings have now been reset.
3. If restarting the pump for an individual, check feeding side “Volume Delivered”. Press **SELECT** until “Volume Delivered” is selected. Press **RESET** to clear to **0**. This will also reset the water side “Volume Delivered” to **0**. Accumulated Volumes is not affected.

Programming Feeding Side:

1. **Feeding Rate**—Use the +/- arrows to set a feeding rate in mL/hr. Maximum feeding rate is 295mL/hr.
2. **Dose Limit**—Use the +/- arrows to program amount of formula (mL) to be delivered. The Dose Limit can also be determined by multiplying the feeding rate by the number of hours the pump should deliver formula.
 - For continuous feeding select none. Pump will deliver formula until you turn it off or container is empty.
3. **Strength**—Keep **FS** for full strength.

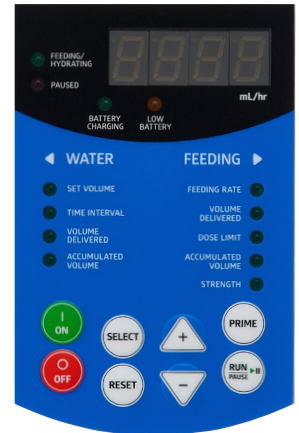

Scan to watch 5-minute instructional video for the control panel.

Programming Water Side—Continuous water delivery with formula + 25 mL automatic quick flush

1. **Set Volume**—Use the +/- arrows to set amount of water (mL) to be delivered over the time interval (hour).
 2. **Time Interval***—Use the +/- arrows to set hour interval, adjustable in 2–12 hour increments. Maximum water delivery is 75 mL/hr. The final 25 mL of water is automatically flushed through the line before the conclusion of the selected time interval.
- *Water Time Intervals repeat as long as the formula side is running.

Programming Water Side—Water bolus* after formula delivery

1. Ensure the Dose Limit is programmed on the Feeding Side.
2. **Set Volume**—Use the +/- arrows to set amount of water (mL) to be delivered after the formula.
3. **Time Interval**—Use the +/- arrows to select Dose.

*Water bolus delivered at 375 mL/hr. Only 1 bolus can be programmed at a time. After formula/water have been delivered pump will alarm “Dose Del”.

Loading the feeding sets:

All feeding sets must be changed every **24 hours**

1. Close both clamps of the feeding sets.
2. Pour liquid into bag/spike container and hang on IV pole.
3. Load feeding sets following diagram. Close both doors by pulling the blue lever over the latch. Do not slam doors.




Auto-Prime:

1. After pump is programmed and set is loaded, unclamp and press **PRIME**.
2. Formula side will prime first, followed by the water.
3. Prime time = approx. 3 minutes. Failure to use the auto-prime will result in alarm errors.



Drip Chambers

Note: The water drip chamber has an extra top piece of plastic. This is a safety feature and door will not close if inserted on feeding side.

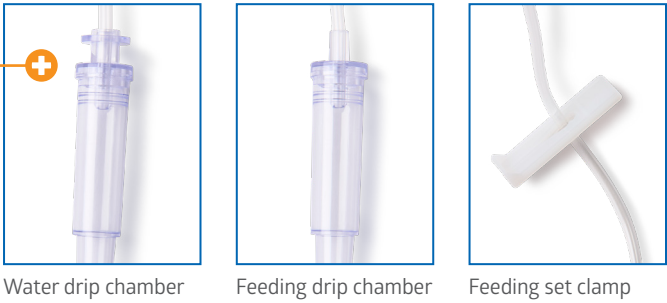

EntraFlo H2O Feeding Pump Tips

Question	Answer
How do I prime?	1. Clamp feed/water tubing BEFORE spiking the container or pouring water/formula into bag*. 2. Load completely DRY drip chambers into the feeding pump and close doors. 3. Once doors are closed, unclamp both sides. 4. Press Prime – approx. 3 minutes to prime. *Failure to clamp sets before spiking bag, manually priming, squeezing the bags to prime faster will result in Feed Err.
Why did the water side not prime?	If “None” is programmed in “Set Volume” or “Time Interval” on the water side, the pump assumes you only want to deliver formula. It will prime formula to the end of the tubing leaving the entire water side filled with air. This is okay if you’re not running water. If you want to run water, you need to program “Set Volume” & “Time interval” to the ordered settings and re-prime the pump. *Always program the feeding pump before priming to avoid Feed Err.
How do I change the feed/water rate?	1. Pause the pump. 2. Press SELECT until “Volume Delivered” is selected . 3. Press RESET to clear “Volume Delivered”. 4. Press SELECT until Feeding Rate* or Set Volume is selected and use the arrows to make the change. *Feeding/water rate changes can only be made at the start of a new cycle. “Volume Delivered” must read “0” in order to make any changes. When resetting the volume delivered on the formula side, it automatically resets volume delivered on the water side and restarts the time interval.
Why am I getting “Feed Err”?	1. Water/formula got into the drip chambers before the pump was primed using auto-prime button. 2. Pump is manually primed. 3. Bottom of feeding set container/bag is less than 6 inches above the feeding pump. 4. Silicone tubing is too stiff/kinked. Re-wrap around the blue rotor. 5. Water side was not primed (and you’re using the water side). Always program the pump BEFORE priming. If “none” is programmed on the water side, “Set Volume” and “Time Interval” the water side will not prime 6. Drip chamber is not fully placed in the holder. 7. Drip chamber walls are coated with drops or condensation in the sensor path. 8. Black flow sensors are dirty. Use alcohol wipe to clean. 9. Feeding set has been in use for longer than 24 hours.

More questions? Email ProductSupportPrimaryCare@medline.com or call 1-800-MEDLINE.

Medline Industries, LP | 1-800-MEDLINE (633-5463) | www.medline.com

© 2024 Medline Industries, LP. All rights reserved. Medline is a registered trademark of Medline Industries, LP. All product names, regardless of whether or not they appear with a trademark symbol, are the property of their respective companies unless otherwise noted. MKT19W5352707 / e16325 / 300 / DP / 94

